




# OPTOMED E-SAFE


Disposable eye cups for hygienic  
retinal imaging

[www.optomed.com](http://www.optomed.com)


# How to use the disposable eye cups

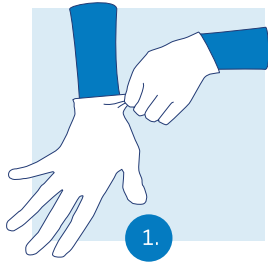


Wear disposable gloves for hygienic handling

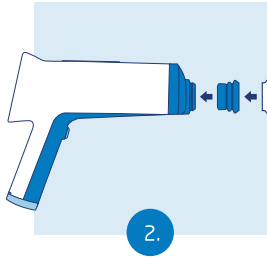


Attach both base and disposable parts to camera

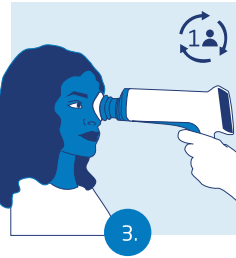

Take the images of the patient

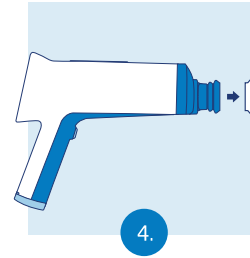

Remove the disposable part after use

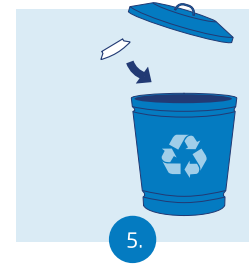

Replace disposable part after each patient

*(Recycle according to local instructions)*


- For single patient use
- 2-piece construction
- Disposable part material: Thermoplastic polyurethane (TPU)
- ~ 1000 disposable parts can be used with one base part

## Available package size:

- Starter pack : 5 pcs base, 200 pcs disposable parts



Base part

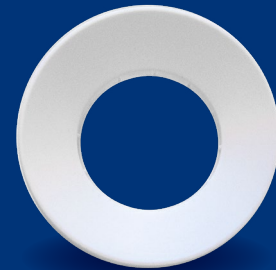

Disposable part